Pascal Handbook
Manuel de la chambre de pression Pascal
Pascal Handbuch

Code No. S2800

Pascal 

For In Vitro Diagnostic Use

CE

Copyright © 2005 DakoCytomation. All rights reserved.

This document may not be copied in whole or in part or reproduced in any other media without the express written permission of DakoCytomation. Please note that under copyright law, copying includes translation into another language.

Table of Contents

Section 1	6
Unpacking and Installation	6
Inspect Packaging	6
Verify Contents	6
Install the Pascal	6
Connect Power	7
Section 2	8
System Overview	8
Principle and Intended Use	8
Symbols and Definitions	8
Audible Indicators and Error Messages	10
Parts Description	10
Section 3	11
Operating Notes	11
Section 4	13
Operating Instructions	13
Set Up	13
Programming	13
Pascal Factory Settings	13
Pascal Factory Settings	13
Pascal Setting Limits	14
Start Program	14
End Program	15
Changing Factory Settings	15
Section 5	17
Maintenance	17
Daily	17
Monthly	17
Pan, Gasket, Vent Nozzle and Weight	17
Heat Plate and Heat Sensor	17
Body	17
Biannually	18
Pascal Calibration Protocol	18
Electrical Precautions	20
Service	20
Disposal	20
Troubleshooting	20
Strong Target Retrieval performance	20
Inconsistent/weak Target Retrieval performance	21
Poor morphology	21
Adherence of specimen to glass slide	21
Specifications	21
DakoCytomation Warranty	22

Limitation of Liability.....	22
Appendix A	23
Pascal Quality Control Log.....	23

How to use this handbook

This handbook, along with information contained on product labels, provides all the information needed to operate and maintain the Pascal pressure chamber. Notes are set off from the handbook text to highlight information.

Please pay close attention to the instructions that accompany the notes as well as the standard laboratory practices outlined by your facility and local regulatory agencies.

Please use the system as intended. Improper use of Pascal may cause damage to the system, inaccurate results, or potentially nullify warranties.

Translations of this handbook are available online at www.dakocytomation.com.

Introduction

The DakoCytomation Pascal pressure chamber is designed to control the parameters of heat and pressure over time. Pascal was specifically engineered to pass strict laboratory safety requirements. Quality control methods are provided to ensure consistent pre-set pressure and temperature levels that can be monitored and recorded.

Pascal Quality Strips

Pascal Quality Strips are heat and pressure sensitive strips that allow the user to monitor conditions inside the Pascal pressure chamber. The strip must register the proper color (charcoal black) to ensure that optimum heat and pressure are achieved. The steam strip can be dated and used as a record for laboratory and CAP inspections. Pascal Quality strips can be reordered as DakoCytomation product code number **S2801**.

Section 1

Unpacking and Installation

Inspect Packaging

Pascal and its accessories are delivered in one carton. If the instrument or accessories have suffered any damage in transport, please inform your carrier immediately.

Note Save shipping carton and components to simplify return in the event that service is required.

Verify Contents

The Package contains:

Lid with Pressure Gauge and Hot Sticker
Weight (Petcock)
Pan and Gasket (removable)
Body with electric cord and programmable digital timer
Heat Shield
Pascal Quality Strips (S2801)

Note Pascal is a 110V–120V instrument. A step-down transformer is required for laboratories using Pascal with a 220V–240V electrical source. This step-down transformer is not supplied within the same box as Pascal.

Install the Pascal

1. Place the body of Pascal on a level surface suitable for laboratory instrumentation.
2. Position Pascal away from direct sunlight and sources of heat or cold.
3. Place the pan in the body. Make sure the outside of the pan is clean and that all packaging has been removed.
4. Fit the rubber gasket in the top of the pan.
5. Place the heat shield in the center of the pan.
6. Place the weight (petcock) on the vent nozzle. Ensure that it is firmly in place.
7. Rest the lid on the pan until ready to use.

Connect Power

Pascal is a 110V–120V instrument. In locations with 110– 120 V supply power, simply plug the instrument into the appropriate outlet.

If the location supplies 220–240 V power, a step down isolation transformer is required (dual winding, rated 1000 VA minimum, 50–60 Hz, 220–240 V input 110–120 V output nominal 13A, with an IEC female output connector). An isolation transformer can be connected with portable cord and plug to the main supply. Pascal is then connected to the isolation transformer.

Do not operate Pascal with other types of high electrical demand equipment such as incubators, microwave ovens, or refrigerators on the same circuit. This may affect the performance of the instrument.

The main power switch is red and is located to the right of the operation panel on the front of the instrument. This switch will illuminate in the **ON** position.

I = ON and **O** = OFF

Section 2

System Overview

Principle and Intended Use

For in vitro diagnostic use. Pascal may be used for Target Retrieval of specimens on glass microscope slides.

Pascal consists of a microprocessor controlled, bench top pressure chamber with lid. Pascal allows for pressurized heating at temperatures above boiling but can also be used at a variety of temperatures ranging from 37 °C to 125 °C

Symbols and Definitions

The following is a list of definitions for symbols and LED indicator lights that appear on the Pascal operation panel located on the front of the instrument.

Display Set (button): Navigates users through the two programs SP1 and SP2. For example, to advance from temperature and time settings for SP1 to temperature and time settings for SP2, press **Display Set**.

Up ▲ and Down ▼ arrow (buttons): Adjusts the settings for temperature or time

Start/Stop (button): Starts and stops instrument based on the set temperature and time functions.

SP1 (LED): Set Point 1 has two functional readouts on the digital display. The first SP1 display is the **temperature** at which Pascal can be set for operational use. The second SP1 display is the **time** interval for which Pascal can be set to maintain the SP1 temperature.

SP2 (LED): Set Point 2 has two functional readouts on the digital display. The first SP2 display is the **temperature** programmed to indicate that the chamber has cooled sufficiently to permit the removal of contents. The second SP2 display is the **time interval** at which the SP2 temperature is maintained before an audible alarm (beep) is sounded. The purpose of the SP2 alarm is to standardize the time at which the contents in Pascal should be removed from the instrument for cooling. This is a quality control function that aids in consistency between runs.

SP Limit (LED): Set Point Limit is an internal temperature set point circuit breaker that ensures the SP1 temperature will not exceed SP1+ SP Limit temperature; for example, if SP1 = 125 °C and SP limit = 10 °C then a temperature of 135 °C will automatically turn off the heating element. This is a safety feature. For example, if an insufficient volume of water is added to the system the temperature could rise beyond the SP1 temperature.

Actual (LED): Indicates that Pascal is in operation and the temperature on the digital display is the current temperature in the chamber in degrees centigrade.

Symbols and Definitions

REF	Product/Reference Number	Indicates the product/catalog number
	Caution	Statement of caution/warning, read instruction carefully
	Temperature limitation	Indicates storage requirements range
EC REP	EC Representative	European Community Authorized Representative
IVD	In vitro diagnostic medical device	Clarifies for use as <i>in vitro</i> diagnostic only
	Non sterile	Indicates non-sterile product
S N	Serial Number	Indicates instrument serial number code
	Consult Instructions	Consult instruction manual for further explanation
	Date of Manufacture	Indicates the date the device was manufactured
	Manufacturer	Indicates the manufacturer identification
	Disposal	Separate collection for electrical and electronic equipment

Audible Indicators and Error Messages

Audible alarm: An audible alarm will sound when Pascal reaches either the programmed SP1 (Set Point 1) time and the SP2 (Set Point 2) temperatures. For quality control purposes, the temperature and pressure can be recorded at these set points.

ERR: These letters will appear on the digital display if the instrument encounters a technical issue such as the following:

The pan is not filled with a sufficient volume of water and therefore the heat and pressure rise beyond normal limits.

The SP Limit may be set too low so that the normal temperature tolerance of the instrument heats beyond the SP1 + SP Limit.

Heat Sensor fails.

Parts Description

Gasket: The rubber seal between the pan and the lid that enables an air-tight chamber. Reorder code number SP99015.

Pan: The removable metal chamber that is placed within the body of Pascal. The pan comes in contact with the heat sensor and the heat plate. Reorder code number SP99019.

Vent Nozzle: The part of the chamber lid where the weight (petcock) should be placed prior to operation of Pascal.

Heat Shield: A circular, concave metal disc with perforations that should be placed in the center of the pan prior to operation of Pascal.

Heat Sensor: A circular, spring-loaded device within the main body of Pascal that records the actual temperature during operation.

Heat Plate: A large, metal disc area within the main body of Pascal surrounding the Heat Sensor that heats the pan during operation.

Section 3

Operating Notes

Pascal has been designed with many safety features unique from other pressure chamber devices; however, when using heating and electrical equipment, basic precautions should be used. This is a heating device under pressure. Please read the operating notes carefully prior to operating the Pascal.

OPEN PASCAL LID CAUTIOUSLY

When there is resistance on the lid, do not open by force. Pressure in the chamber may make the lid difficult to open. Tip the weight carefully to release any residual pressure and make sure that the pressure in the pan is at zero on the pressure gauge. Gently push down on the brown handle, turn the lid counter clockwise, and slowly lift off the lid.

DO NOT OPERATE WITHOUT WATER IN THE PAN

**** VERY IMPORTANT**** Make sure there is deionized water in the pan. Do not turn on the heat when the pan is empty. This can cause malfunction and jeopardizes the integrity of the instrument.

CHECK THE WEIGHT PRIOR TO EACH USE

Make sure the weight (petcock) located on the vent nozzle is secure.

HANDLE WITH CARE

Dropping or roughly jolting Pascal will cause damage and deformation leading to malfunctions.

DO NOT OPERATE NEAR A HEAT, FLAME OR GAS SOURCE

Never place the Pascal directly on a stove or other heating sources.

KEEP PASCAL CHAMBER BODY DRY

Remove foreign matter and water drops inside the body and on the outside of the chamber pan. The heating plate, the sensor (inside the body) and the outside of the pan must be dry and clean in order to work properly.

COOL DOWN AFTER EACH USE

In order to maintain consistency between runs, Pascal should be started at the same initial temperature for each run. This can be done in a variety of ways. The user can cool the unit down by adding cool water to the pan to bring the temperature down to 37 °C, for example. Alternatively, the instrument can be brought up to 80 °C by setting SP1 to 80 °C. When the alarm sounds, the user can load the instrument, program the run parameters, and start the run. This ensures consistency between runs. If only one run is performed per day then this may not be an issue since the starting point will be close to the same each day, i.e. ambient temperature.

CHECK THE GASKET PERIODICALLY

Put the gasket check for Pascal on your monthly preventative maintenance list. The gasket must be checked consistently for tears or damage. A replacement gasket can be ordered with DakoCytomation code number **SP99015**. A relatively weak Pascal Quality Strip color may indicate the need for a new Pascal gasket.

FILL THE UNIT WITH ONLY 500 mL FOR MOST APPLICATIONS.

Fill Pascal with deionized water. 500 mL is all that is required for most applications.

AVOID TOUCHING THE PAN OR LID WHEN IN USE – HOT!

When in use, the pan, the lid and the weight get very hot. Avoid touching them. There is a hot sticker to warn that the lid is hot. The circular sticker reads **HOT at elevated temperatures**.

Section 4

Operating Instructions

Set Up

1. Familiarize yourself with Pascal and its parts prior to operating.
 2. Adjust the pan by matching the pan handles with the body handles.
- Note** If the outside of the pan is wet, Pascal will make a cracking noise. Any water in the body may cause malfunctions.
3. Fill the pan with 500 mL of deionized water for performing the standard Target Retrieval procedure.
 4. Ensure the heat shield is in the center of the pan.
 5. Place 1 to 4 Tissue-Tek[®] containers each filled with 250 mL of suitable Target Retrieval Solution in the pan. Alternatively, use plastic Coplin jars. Fill each plastic Coplin jar with 50 mL of Target Retrieval Solution. Place slides intended for the Target Retrieval procedure in the Tissue-Tek[®] containers or Coplin jars.
 6. Place a dry Pascal Quality Strip on top of the containers.
 7. Position the lid on the pan and secure in place by aligning the **OPEN** arrow etched on the lid's surface with one of the white dots on the pan handle. Grip the lid handle and rotate clockwise so that the **CLOSE** arrow is aligned with the white dot on the pan handle.
 8. Place the weight (petcock) on the vent nozzle located on the Pascal chamber lid.
 9. Turn the red main power switch to the **ON** position (**I**).

Programming

Pascal Factory Settings

Push the **Display Set** button to recall the following factory setting parameters on a new instrument. These settings can be changed for individual applications. Refer to Changing Factory Setting within this section for notes on making changes for specific applications.

Pascal Factory Settings

Factory Settings	Temperature	Time
SP1	125 °C	30 seconds
SP2	90 °C	10 Seconds
SP Limit	10 °C	n/a

Pascal Setting Limits

	Minimum	Maximum
SP Temperature	37 °C	125 °C
SP Time	:01 second	98:59 minutes

These are the factory settings. SP1 and SP2 settings can be changed by the user. DakoCytomation recommends performing Target Retrieval in Pascal at 120–125 °C for 30 seconds–5 minutes. Recommended times may vary according to the application. Most DakoCytomation applications recommend performing Target Retrieval in a pressure chamber set at 125 °C for 30 seconds.

Note The Pascal Quality Strip optimal performance (charcoal black color) is at 125 °C and relative pressure. Lower temperature and pressure will produce a lighter colored strip; this should be consistent at the same temperature and pressure between runs.

Start Program

1. Press the **Display Set** button until the SP1 LED indicator illuminates.
2. Push the **Start/Stop** button to start the program.
3. Push the down arrow to display the actual temperature.

Note Both the **Heat On** light and **Actual** temperature light should be illuminated. The time required to reach the temperature programmed depends on many variables such as the voltage source and frequency, the number of bath/slides in the chamber, and the altitude at which the instrument is operated.

Note Upon reaching the programmed temperature, the **Heat On** LED will turn off and the timer will start counting down. The instrument will overshoot the programmed temperature by approximately 2–3 °C, and then will slowly come down to the programmed temperature. When the digital timer reaches 0, the audible alarm will sound.

4. When the alarm sounds, press the **Start/Stop** button and the **SP2** LED will illuminate.
5. For quality control, immediately record the temperature and pressure.

Note For quality control, immediately record the temperature and pressure. The pressure will vary with contents within Pascal and the SP1 temperature setting. The temperature may read +0–3 °C above set SP1 temperature setting.

End Program

1. When the temperature reaches the SP2 programmed temperature for the programmed SP2 time limit, an audible beep will sound and all indicator-lights will be flashing. Press the **Start/Stop** button to end the program. Confirm the pressure reads 0 psi, before unlocking the Pascal lid.

Note If pressure is not at 0 psi when the SP2 audible alarm is reached but is low relative to the pressure reached during the run, i.e. approximately 1-5 psi, the user should gently tilt the weight (petcock) located on the lid to one side. This will release residual pressure and enable a safe environment in which to open Pascal post run.

2. Slightly push down the lid and rotate counter-clockwise to the **OPEN** position. Turn the **On/Off** switch to the **Off** position.

3. Check the Pascal Quality Strip for proper color (dark gray to charcoal black) and record for quality control.

4. Remove container with slides and slowly rinse slides in running reagent grade water. Slides should remain in the container while rinsing.

Note In order to maintain consistency between runs, Pascal should be started at the same initial temperature for each run. This can be done in a variety of ways. The unit can be cooled down by adding cool water to the pan to bring the temperature down to 37 °C, for example. Alternatively, the instrument can be brought up to 80 °C by setting SP1 to 80 °C. When the alarm sounds, load the instrument, program the run parameters and start the run. This ensures consistency between runs. If only one run is performed per day, this may not be an issue because the starting point will be close to the same each day, i.e. ambient temperature.

Changing Factory Settings

1. Press the **Display Set** button until the **SP1** LED illuminates. Use the up and down arrows to set the SP1 **temperature** to the desired temperature. SP1 can be utilized exclusively when the SP2 time is programmed at 0 minutes.

Note To speed up the time and temperature setting, push the second arrow while holding the original direction arrow down.

Note Time Bypass feature: if Set Point 1 (SP1) time is programmed to 99:00 minutes, Pascal will continuously maintain temperature without a time countdown. When using the Time Bypass feature, the temperature cannot be set at greater than 100 °C as a safety precaution. If the bypass feature is set at 99:00 minutes at SP1, the SP2 function is disabled.

2. Press the **Display Set** button again and use the up and down arrows to set the SP1 **time** to the desired program time.

3. Press the **Display Set** button again until the **SP2** LED illuminates. Use the up and down arrows to set the SP2 **temperature** to the desired cool down temperature.

4. Press the **Display Set** button again and use the up and down arrows to set the SP2 **time** to the desired program time.

Note When Pascal has reached the SP1 temperature and time, the heating element will turn off automatically and cannot be turned on again until after reaching an internal temperature of 37 °C. Also, the SP2 temperature will not be maintained when reached; the alarm is only an indicator that the SP2 temperature has been reached.

5. Press the **Display Set** button until the **SP Limit** LED indicator illuminates. Use the up and down arrows to set the **SP Limit** to the desired circuit breaker temperature setting.

Section 5

Maintenance

Daily

The following maintenance procedures should be performed after each daily use of Pascal:

Pour water out of pan.
Rinse pan with deionized water and dry.
Inspect gasket for wear.
Lid can be left on top of pan, but should be left slightly ajar.

Monthly

Preventative Maintenance should be performed once per month:

Pan, Gasket, Vent Nozzle and Weight

- Remove the pan, gasket, lid, and weight and soak each in warm water with a mild dish detergent. Abrasive powders or metal scrubbers will damage the surface and may cause contamination. Wash thoroughly with a sponge and then dry well. To protect against electrical hazards, do not immerse cord/plug or the body of Pascal in water or a dishwasher.

Inspect gasket for wear. Do not pull or tear the gasket.

Inspect lid and pan for corrosion - especially where these parts come into contact with the gasket.

If the vent nozzle becomes blocked with debris, clear the path with a toothpick or similar device and wash clean.

Heat Plate and Heat Sensor

Wipe off anything stuck to the heat plate or heat sensor. Dry with a soft cloth.

Body

Wipe the surface lightly with a soft cloth soaked in warm water.

Be careful that water does not get into the body.

Biannually

Pascal Calibration Protocol

Pascal calibration is performed at two temperatures. Both calibration points must be performed for a complete Pascal calibration procedure. While the complete calibration procedure is recommended every 6 months, it is also recommended to perform the room temperature calibration prior to each use

Room Temperature Calibration

1. Plug Pascal into power supply
 - Turn on the power switch located on the right side, next to the front panel.
2. Place 500 mL water into Pascal pan and equilibrate to room temperature for 15–20 minutes.
 - This step is necessary in order to validate the starting temperature.
 - Do not add warm water to speed up the process.
3. Read the temperature on the digital display.
4. Insert a calibrated thermometer in the Pascal pan containing 500 mL of water. Compare the thermometer temperature reading to the digital display reading. If the digital display matches the thermometer reading within 0.5 °C then the instrument should be considered within calibration limits for the Room Temperature Calibration check point. Proceed to the High Temperature Calibration.

If the readings fail to match by greater than 0.5 °C, adjust the digital reading by following these steps:

- Turn the main power switch off.
- Turn it back on while pressing the Up ▲ and Down ▼ arrows at the same time.
- Confirm that all 6 LEDs are illuminated; this indicates that Pascal is in calibration offset mode.
- Press the “Display Set” button one time.
- Use the Up ▲ or Down ▼ arrow to adjust the offset so the digital temperature read out will match the calibrated thermometer. Example: The digital reading is 25 °C, and the thermometer reading is 26 °C. The action required is to adjust the offset up by 1 °C.
- Once the calibration is set press the “Display set” button two times to exit the calibration mode.

High Temperature Calibration

1. Ensure that Pascal is set to factory settings for SP1, SP2, and SP Limit for the High Temperature Calibration procedure. If Pascal settings have been changed from factory specified settings refer to the Operating Instructions section in this

manual for both the factory setting parameters and the instructions on changing Pascal settings.

2. Place 500 mL water inside the Pascal pan followed by a 250 mL bath of water containing a slide rack with 24 slides.
 - Place a Steam Monitor Strip on the bath container.
 - Close the Pascal lid and press the “Start/Stop” button.

3. When the SP1 temperature is reached and the alarm sounds verify the digital display temperature is 126–128 °C and chamber pressure is 21–23.4 pounds per square inch (psi). If these parameters are within this range then the Pascal should be considered to be within calibration limits for the High Temperature Calibration check point. If the temperature and the pressure are not within the above range, perform the following adjustment:
 - Turn the power switch to the off position.
 - Turn it back on while pressing the Up ▲ and Down ▼ arrows at the same time.
 - Confirm all 6 LEDs are illuminated; this indicates that Pascal is in calibration offset mode.
 - Press the “Display Set” button one time.
 - Adjust the digital display temperature reading by using Up ▲ or Down ▼ arrows to match the pressure that is displayed on the Pascal pressure gauge to the corresponding temperature recorded on the Pressure-Temperature Calibration Chart below.
 - **Note** The arrow button may need to be pushed and held with consistent pressure until the temperature changes. Work quickly as the temperature and pressure are decreasing.

Pressure-Temperature Calibration Chart

Pressure	Temperature	Pressure	Temperature
13.0 psi	118.0 °C	18.9 psi	124.0 °C
13.5 psi	118.5 °C	19.5 psi	124.5 °C
14.0 psi	119.0 °C	20.0 psi	125.0 °C
14.4 psi	119.5 °C	20.5 psi	125.5 °C
15.0 psi	120.0 °C	21.0 psi	126.0 °C
15.4 psi	120.5 °C	21.6 psi	126.5 °C
15.9 psi	121.0 °C	22.2 psi	127.0 °C
16.3 psi	121.5 °C	22.8 psi	127.5 °C
16.8 psi	122.0 °C	23.4 psi	128.0 °C
17.3 psi	122.5 °C	23.9 psi	128.5 °C
17.8 psi	123.0 °C	24.5 psi	129.0 °C
18.3 psi	123.5 °C	25.1 psi	129.5 °C

One pound force per square inch (1 psi) equates to 6.895 kPa (kiloPascals), conversion factor provided by United Kingdom National Physical Laboratory.

4. Turn the Power off, and wait for Pascal to cool down to room temperature, i.e. <27 °C. Cold water can be used to reduce the device’s temperature for this step.

5. Repeat the High Temperature Calibration to verify that Pascal now falls within the calibration limits specified in step 3 of the High Temperature Calibration procedure.

Electrical Precautions

Basic safety precautions should always be followed when using electrical equipment.

Read the following instructions:

- To protect against electrical hazards, do not immerse cord/plug or the body of Pascal in water or a dishwasher.
- Unplug from outlet when not in use, and before cleaning.
- Do not operate Pascal with a damaged cord or plug.
- Do not let cord hang over edge of table or counter, or touch hot surfaces.
- Do not place near a heat source, open flame or gas outlet.

Service

There are few user-serviceable parts such as the gasket, lid and pan. Refer all service to qualified DakoCytomation service personnel.

Reference the DakoCytomation Warranty for further instructions.

Note Decontamination before returning for service is required.

A Pascal containing accumulated blood and/or other biological or chemical deposits must be cleaned prior to shipment to the manufacturer/dealer for service. Refer to local or regional regulations for decontamination requirements. DakoCytomation personnel cannot perform this decontamination.

Disposal

Correct Disposal of this Product

(According to *Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE)* applicable in the European Union and other European countries with separate collection systems).

Contact DakoCytomation representative for disposal of the equipment at the end of its working life. This product should not be mixed with other commercial waste for disposal.

Troubleshooting

Strong Target Retrieval performance

- Reduce programmed time interval, i.e. if set at 5 minutes for a given temperature, reduce time to 30 seconds to decrease stain intensity.
- Alternatively, if time interval is already short, reduce programmed temperature.
- Decrease antibody incubation time.
- Increase antibody dilution.

Inconsistent/weak Target Retrieval performance

Ensure the integrity of the gasket. Check to see if pressure is consistent between at least three separate runs. If pressure is dropping relative to standard operation based on a consistent program setting, replace the gasket.

Poor morphology

- Reduce programmed time interval; i.e., if set at 5 minutes for a given temperature, reduce time to 30 seconds to decrease time at high temperature which may affect morphology within certain tissue types.
- Use appropriate Target Retrieval Solutions. High pH buffers (i.e. pH 9.9) have a tendency to be harsh on tissue specimens and can contribute to poor morphology.

Adherence of specimen to glass slide

Start retrieval program at a higher temperature; i.e., 80 °C as opposed to lower temperature such as ambient or 37 °C. This has been shown to reduce tissue adherence failure.

Use appropriate Target Retrieval Solutions. High pH buffers have a tendency to be harsh on tissue specimens and can contribute to tissue adherence failure.

Ensure tissue sections are mounted on silanized slides or (+) charged slides.

Ensure that tissue sections were mounted from a waterbath not containing gelatin or other adhesive product.

Ensure that sectioned slides have been baked for 1 hour at 60 °C before processing.

Specifications

Model: S2800 (110–120 volts)

Pan Capacity: 3.2 liters

Weight: 3.7 kg (8.18 lbs.)

CE Listed

DakoCytomation Warranty

DakoCytomation warrants to Buyer that Pascal will be free, under normal use and maintenance, from defects in material and workmanship for a period of one (1) year from the date of delivery to the end-user. As Buyer's sole and exclusive remedy for any such defects, DakoCytomation will, at its option, repair or replace without charge any unit which is found to be so defective.

Limitation of Liability

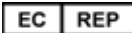
DakoCytomation shall not be responsible under this warranty for products found to be defective as a result of misuse, mishandling, or neglect.

DakoCytomation MAKES NO OTHER WARRANTIES OTHER THAN THAT SPECIFIED ABOVE, OF ANY KIND WHATSOEVER, EXPRESS OR IMPLIED, AND ALL WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE HEREBY DISCLAIMED BY DAKOCYTOMATION AND EXCLUDED FROM THE TERMS OF SALE.

DakoCytomation, Inc.
6392 Via Real
Carpinteria, California 93013 USA

Tel 805 566 6655
Fax 805 566 6688
Technical Support 800 424 0021
Customer Service 800 235 5763

DakoCytomation Denmark A/S
Produktionsvej 42
DK-2600 Glostrup Denmark

Tel +45 4485 9500
Fax +45 4485 9595

www.dakocytomation.com

Appendix A

Pascal Quality Control Log

Use the following quality control log to record settings such as pressure and temperature to ensure consistent performance of the Pascal.

Pascal Quality Control Log

mm/dd/yy - mm/dd/yy
 Log Dates _____ - _____

Operated By: _____

Run No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
Technician Initials															
Run Date															

Operation Parameters*

Volume Water in Pan (mL)															
Room Temperature Calibration Checked (°C)															
SP1 Temperature Set Point (°C)															
SP1 Time Set Point (minutes/seconds)															

Operation Results

SP1 Temperature (°C)															
SP1 Pressure (psi)															
Pascal Quality Strips**															

Pascal Quality Strip (attach)***															
----------------------------------	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

Reviewed and Approved by : _____

Date: _____

Notes

- *Operation Parameters are user defined programmed times and temperatures based on user's protocol(s).
- **Enter 'D' for DARK, 'M' for MEDIUM, and 'L' for LIGHT. Shade of strip is determined by visual inspection and relative to previous runs/technician experience.
- ***Adhere the run specific Quality Strip under the appropriate run log details.

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Copyright © 2005 DakoCytomation. Tous droits réservés.
Copyright © 2005 DakoCytomation. Alle Rechte vorbehalten.